**EN\_ IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.**

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- A - Backrest
- B - Locking Handle Brakes
- C - Height Adjustable Handles
- D - Seat
- E - Shoulder Bag
- F - Rear Folding Pins (2)
- G - Fastening Knobs (2)
- H - Rear Wheels
- I - Front Wheels

1. Remove all packaging. The rollator will be folded as shown in Figure 1.

Figure 1

2. Unfold the front wheels and secure in place using the fastening knobs as shown in Figure 2.

Figure 2

Fastening Knob

3. Fold out the rear legs as shown below and secure the legs in place with the rear folding pins as shown in Figure 3.

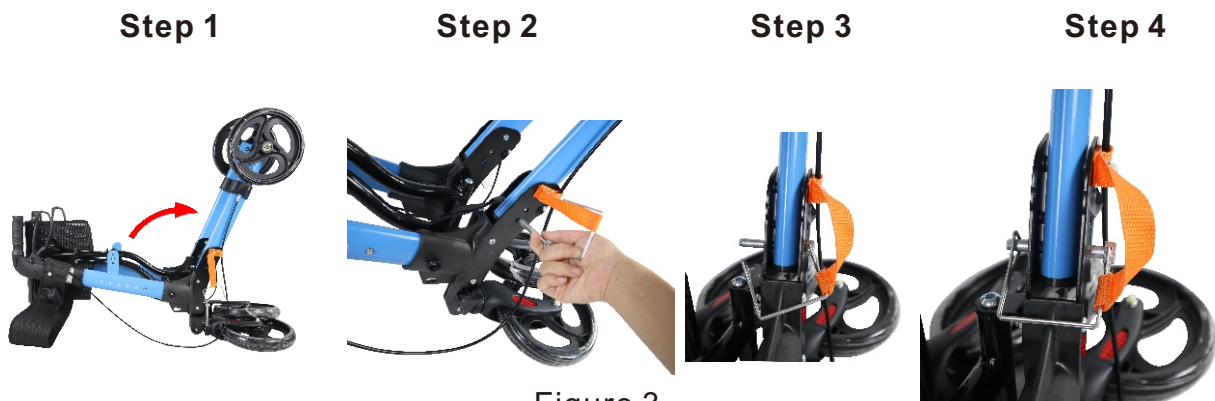
Figure 3

4. Open the rollator, push down on either side of the seat as shown in Figure 4. **IMPORTANT:** Both sides must “click” into position.

Figure 4

5. Attach the shoulder bag.

## Height adjustment

Press the height adjustment button on the auxiliary driving frame to adjust the handle to the appropriate height. Repeat on the other side. Ensure that the buttons on both handrails are locked. Ensure that both handles are set to the same height

Adjustable height of handle: 79.5-93cm

## BRAKE

1. Pull the brake handle upwards to assist with the driving brake

2. Push down on the brake handle to lock the rear wheels, and at this time, the auxiliary vehicle is in the parking state. To release the parking status again, pull the brake lever upwards to release the rear wheels

The brake adjustment is as follows:  
Turn the adjusting screw counterclockwise to increase tension, then turn the adjusting lock nut clockwise while keeping the adjusting screw stationary. Reducing brake pressure may affect the brakes in the locked position.

Movable crutch holder  
Slide the crutch holder down and fix it at the designated position on the side of the frame

#### Folding assisted driving

1. Pull up the pull rope in the middle of the seat

2. Continue to promote the framework on both sides of the auxiliary driving vehicle

#### Safety instructions

When using a walking aid, especially on slopes, sloping surfaces, steps, and shoulders, it is important to always ensure your safety.

Please choose the route carefully and try to avoid uneven or soft roads. If there is snow, ice, a layer of water or oil on the slope, do not use a walking aid.

You must not make any unauthorized adjustments or changes to your walker, as this may cause it to become unstable. You can practice on flat ground or slowly climb and descend slopes

Only one person can use the walker normally. When using, do not stand on the frame, seat, or wheels. Unprofessional or incorrect use or modification may increase the risk of accidents. Users may put themselves in danger due to improper use. When using, the handbrake should be used to slow down the speed of the walking aid. It is important to remember to use the parking brake when parking, especially when storing the walking aid for a long time, otherwise it may damage or stretch the brake line.

If any damage occurs or repairs are needed, immediately stop using the walking aid.

Before use, it is important to ensure that the walker is fully installed and carefully checked.

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If you have any questions, please contact our customer care center, Our contact details are below:

**US**

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC  
7777 Center Ave. Suite 430  
Huntington Beach, CA 92647-9998 USA  
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

**CA**

 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.  
7270 Woodbine Avenue, Unit 307, Markham,  
Ontario Canada L3R 4B9  
MADE IN CHINA

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If you have any questions, please contact our customer care center, Our contact details are below:

**UK**

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:  
MH STAR UK Ltd  
1 Northampton Cross Logistics Park  
Northampton Cross Road  
Northampton  
NN4 9FH  
MADE IN CHINA

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MH Handel GmbH