IN231000805V01

712-056V00

EN_ IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

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LIST OF COMPONENTS

1. One front wheel
2. main frame
3. left handle bar and wheel
4. right handle bar and wheel
5. shopping bag

Your roller is equipped with various components, You should know these components and parts before proceeding to read this user manual. Design and specifications could be changed without further notices.

Overall width: 23 in/ 58 cm
Overall depth: 27.6 in/ 70 cm
Handle height: 33.9-38.8 in/ 86-98.6 cm
Max. user's weight: 300 lbs/136 kg
Net weight: 11 lbs/ 5.17 kg

ASSEMBLY INSTRUCTION

1. Take out all packages.

2. Take out the main frame and open it till the black connecting line straight.

3. Step one, press the red button at the bottom. Insert the left handle bar into the main frame.
Step two, press the spring button down then insert the left leg into the main frame.

4. Step one, press the red button at the bottom. Insert the right handle bar into the main frame. Step two, press the spring button down then insert the right leg into the main frame.

5. Insert the front wheel into front tube then use star screw to fix them. Make sure the screw head fit the regular hexagonal hole.

6. Hang up the bag

HANDLE ADJUSTABLE

The rollator's handles can be adjusted in height to suit different users. Simply depress the spring button and set it to the desired height. Repeat for the opposite side and make sure that both handles are secure and set at the same height before use.

BRAKE OPERATION ADJUSTMENT

1. Pull up on brake handle lever to slow the roller
2. Push down on the brake handle lever to lock the rear wheels, pull up on the brake lever handle to release the lock
3. Brakes must be applied in the lock position when the seat is going to be used. pre check the lock by pushing forward on the roller.

Brake adjustment should be carried out as follows:

Turn the adjusting screw anti-clockwise to increase the tension then lock the adjustment locking nut clockwise whilst holding the adjuster screw stationary. Brake adjustment reduced to allow light pressure may be ineffective on the brake position.

MAINTENANCE

Over a period of time, the brake cables may stretch, and if no further adjustments can be made, the cables should be replaced. Ask qualified technician for help when replacing brake cables. The castors should be checked that they are secure each time before using the roller. Also, check the brakes operate correctly and the handles and hand knobs are secure. Should any part of the roller appear loose or insecure discontinue use until it is inspected by an qualified technician. The following shows the maintenance schedule:

Weekly

- Check brake for malfunction
- Check all wheels and castors for loose bolts and nuts and knobs.

Periodically

- Check hand grips for wear
- Check frame bolts are secure
- Check front and rear axle bolts are secure
- Check for wear or crack

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If you have any questions, please contact our customer care center, Our contact details are below:

US

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC
7777 Center Ave. Suite 430
Huntington Beach, CA 92647-9998 USA
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

CA

 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.
Unit 33-1 Sparks Avenue, North York, Ontario,
Canada
M2H 2W1
MADE IN CHINA

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If you have any questions, please contact our customer care center, Our contact details are below:

UK

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:
MH STAR UK LTD
1 Northampton Cross Logistics Park
Northampton Cross Road
Northampton
Nn4 9FH
Tel: 0800 240 4050
enquiries@mhstar.co.uk
MADE IN CHINA

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MH Handel GmbH