IN231100579V01

**712-057V00**

**EN\_ IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.**

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## INTRODUCTION

The rollators are manufactured from high-quality materials and include a seat, bag, and locking brakes. The hand-bars are adjustable in height and the rollator folds easily for transport or storage.

## INTENDED USE

Designed to assist the user to walk, rest and carry small items of shopping for both indoor and outdoor use on flat surfaces by a single individual with limited mobility with a maximum weight as shown. The intended user should possess the necessary physical, visual, and cognitive skills to operate the rollator safely. It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the rollator or surrounding property.

## SAFETY ADVICE

This rollator is designed to be used as a mobility aid, creating a natural gate as the user walks securely with their primary weight between the handle bars with the unit in front of the user. Read this manual carefully before using your rollator.

## DO

- (1) Always ensure the brakes are in the locked position if the rollator is left unattended
- (2) Check all castors and wheels are secure before use
- (3) Check the brakes are functioning properly
- (4) Check handle height adjustment, hand knobs are secure before Use
- (5) Check the rollator is fully unfolded
- (6) Take care when carrying the rollator when folded.
- (7) Keep your hands away from the folding mechanism to prevent any risk of pinching

## DO NOT

- (1) Always walking with the rollator in a smooth, controlled motion. Seek assistance if necessary.
- (2) Hang objects on the brake levers or hand-grips. This could affect brake operation and/or stability of the rollator.

Every care has been taken to ensure that your rollator will provide safe and reliable service as you adhere to the advice given above.

## CLEANING

Your rollator can be cleaned with a damp cloth or sponge and warm soapy water. Domestic anti-bacterial surface cleaners may be used if required. Care should be taken not to leave any cleaning residues on the hand grips or seat (where fitted) which may leave them slippery when used. Do not use a scouring pad for scrapers this may damage surface coatings.

## WARNINGS!

1. Maximum working limit 136 kg/ 300 lbs.
2. Do NOT stand on this product.
3. Ensure both push handles are adjusted to the same height
4. Seek advice from a trained health care professional for height adjustment of this product.
5. Always maintain a safe distance between your rollator and other pedestrians or objects.
6. Some components may cause harm or damage when they come into contact with other objects.
7. Do not attempt to replace the wheels or castors of your rollator. If they are faulty or require a replacement please contact a qualified technician.
8. Ensure that your rollator is opened fully (folding scissor arm is straight) before reusing it.
9. Ensure that your fingers are clear from the scissor arm when folding

## LIST OF COMPONENTS

1. One front wheel
2. main frame
3. left handle bar and wheel
4. right handle bar and wheel
5. shopping bag

Your rollator is equipped with various components, You should know these components and parts before proceeding to read this user manual. Design and specifications could be changed without further notices.

Overall width:62.5 cm  
Total depth:71 cm  
Handle height:84.5-97cm  
Max. user' s weight:136 kg  
Net weight:5.2kg

## ASSEMBLY INSTRUCTION

1. Take out all packages.

2. Take out the main frame and open it till the black connecting line straight.

3. Press and hold the red button and insert the left handle into the frame.

4. First, remove the screws and nuts from the rear wheel. Step two, insert the rear wheel tube into the main frame. Then insert the screw and nut and turn the nut clockwise until it locks

5. Repeat steps 3 and 4 when installing the right handle and rear wheel

6. Step one, take out the plum knob which plugs into the front wheel. Second, insert the front wheel into the front wheel tube on the main frame, then insert the plum knob, turning the plum knob clockwise until it locks

7. Hang up the bag

#### HANDLE ADJUSTABLE

The rollator's handles can be adjusted in height to suit different users. Simply depress the spring button and set it to the desired height. Repeat for the opposite side and make sure that both handles are secure and set at the same height before use.

## **BRAKE OPERATION ADJUSTMENT**

1. Pull up on brake handle lever to slow the roller
2. Push down on the brake handle lever to lock the rear wheels, pull up on the brake lever handle to release the lock
3. Brakes must be applied in the lock position when the seat is going to be used. pre check the lock by pushing forward on the roller.

### **Brake adjustment should be carried out as follows:**

Turn the adjusting screw anti-clockwise to increase the tension then lock the adjustment locking nut clockwise whilst holding the adjuster screw stationary. Brake adjustment reduced to allow light pressure may be ineffective on the brake position.

## **MAINTENANCE**

Over a period of time, the brake cables may stretch, and if no further adjustments can be made, the cables should be replaced. Ask qualified technician for help when replacing brake cables. The castors should be checked that they are secure each time before using the roller. Also, check the brakes operate correctly and the handles and hand knobs are secure. Should any part of the roller appear loose or insecure discontinue use until it is inspected by an qualified technician. The following shows the maintenance schedule:

### **Weekly**

- Check brake for malfunction
- Check all wheels and castors for loose bolts and nuts and knobs.

### **Periodically**

- Check hand grips for wear
- Check frame bolts are secure
- Check front and rear axle bolts are secure
- Check for wear or crack

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If you have any questions, please contact our customer care center, Our contact details are below:

**US**

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC  
7777 Center Ave. Suite 430  
Huntington Beach, CA 92647-9998 USA  
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

**UK**

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:  
MH STAR UK Ltd  
1 Northampton Cross Logistics Park  
Northampton Cross Road  
Northampton  
NN4 9FH  
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

**CA**

 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.  
7270 Woodbine Avenue, Unit 307, Markham,  
Ontario Canada L3R 4B9  
MADE IN CHINA

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