IN240700928V01

712-035

6

STEPS

0.5 HR

EN_ IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

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EN_Maintenance

1. The exterior of this rollator walker should be cleaned periodically with a soft damp cloth. Do not use abrasive cleaners or caustic chemicals.
2. Check the parts for any looseness, wear, or damage periodically. If any problems are found, correct them before using them.
3. If haven't used it for a long time, please store it in a cool, dry and safe place.

Warning

1. Before using, make sure the rollator is fully opened.
2. Checking the brakes and seat are locked, seat part especially.
3. The brakes must be locked before using the seat.

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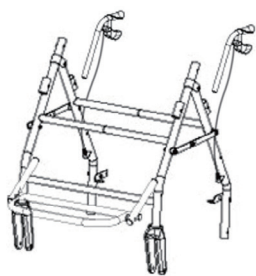
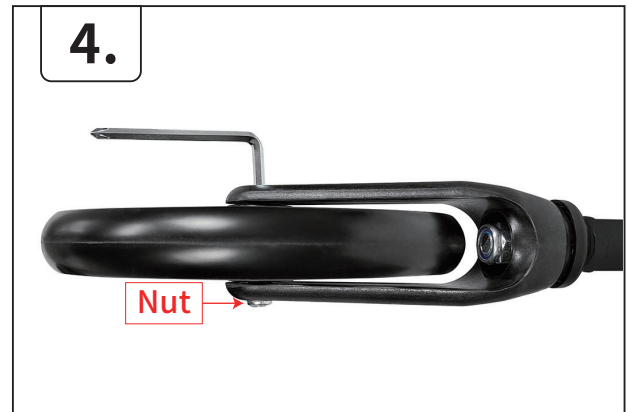
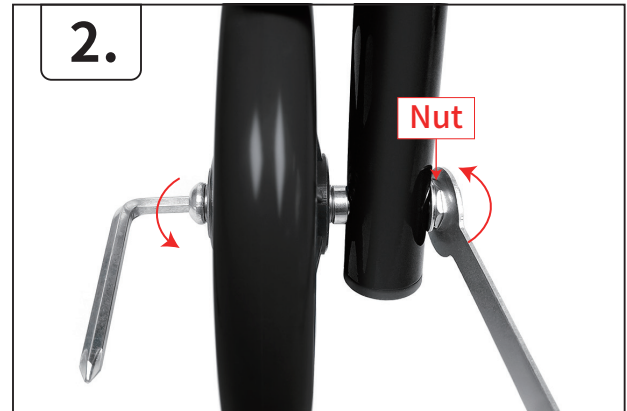
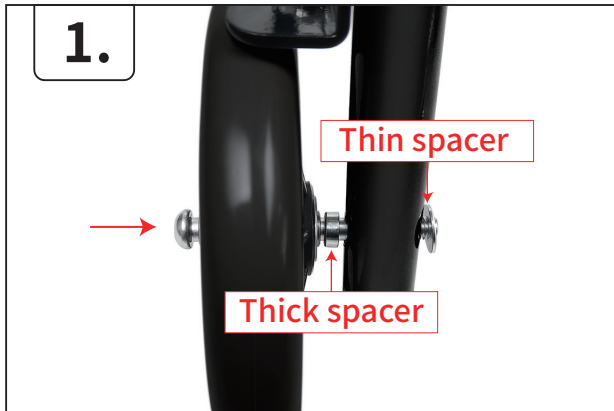
F x 1

① x 1 ⑤ x 4 E x 1 F x 1

EN Install the wheels with the mainframe and secure it with the installation tools .

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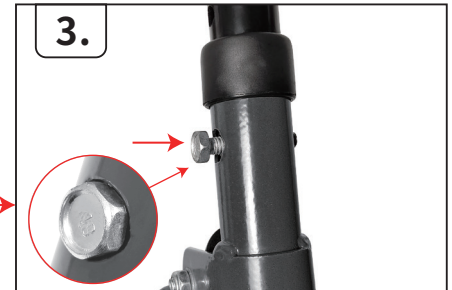
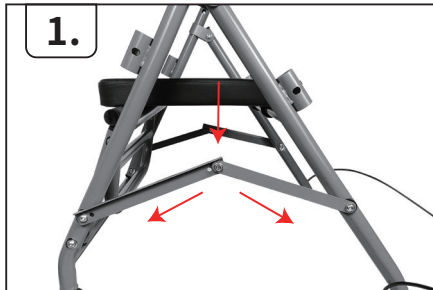
A x 2

C x 2

EN Unfold the mainframe then install the handrail with the mainframe and secure it with the bolt(A) and star nut(C) for both sides and adjust the same height desired, then attach the brake cable clip with the mainframe.

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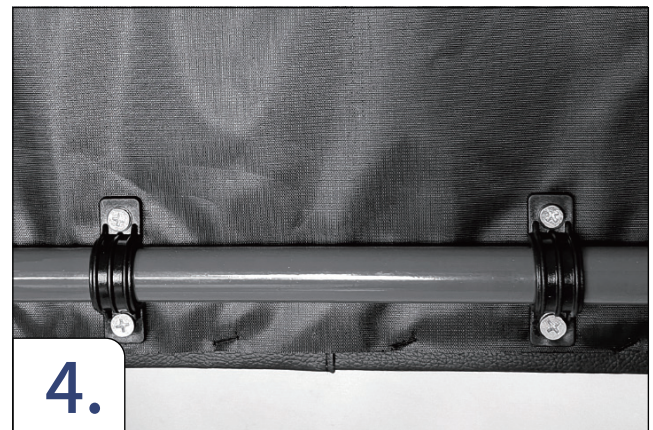
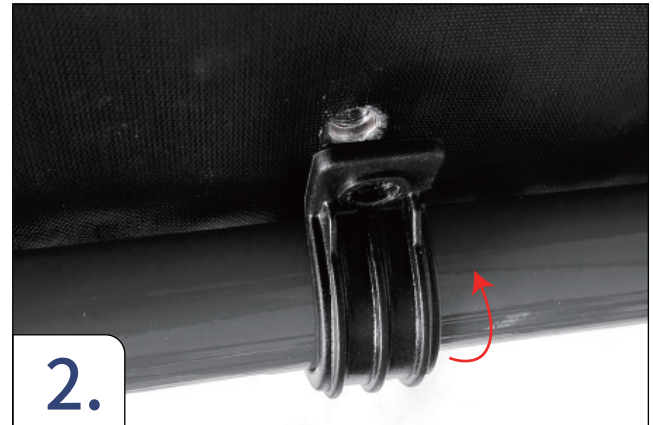
D x 2

F x 1

EN Install the seat cushion with the mainframe and using the installation tools to secure it with the screw (B) and clip (D) as showed .

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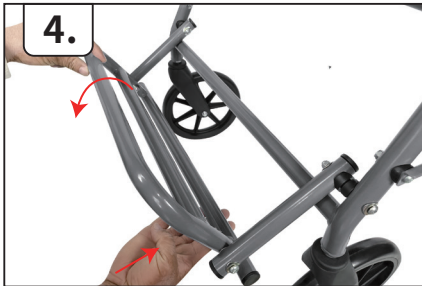
EN Roll over the footrest part of the front wheel body step by step as showed .

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4 x 1

EN Place the basket under the seat as showed .

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3 x 1

EN_Install the backrest with the mainframe till the marble is secured .

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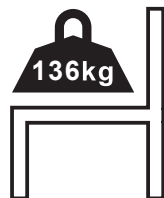
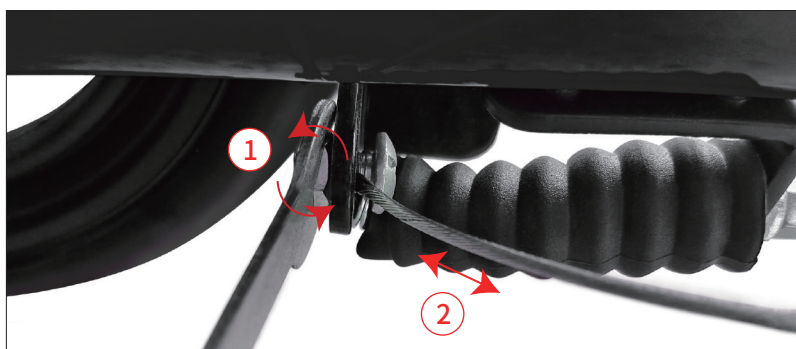
EN_Please adjust the brake cable as showed if you find trouble with the brake system .

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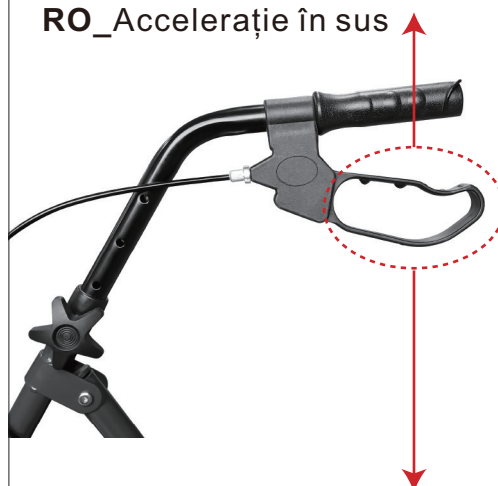
PT_Desacelerar para cima

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EN_Downward stop

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If you have any questions, please contact our customer care center, Our contact details are below:

US

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC
7777 Center Ave. Suite 430
Huntington Beach, CA 92647-9998 USA
MADE IN CHINA

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If you have any questions, please contact our customer care center, Our contact details are below:

CA

 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.
7270 Woodbine Avenue, Unit 307, Markham,
Ontario Canada L3R 4B9
MADE IN CHINA

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If you have any questions, please contact our customer care center, Our contact details are below:

UK

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:
MH STAR UK Ltd
1 Northampton Cross Logistics Park
Northampton Cross Road
Northampton
NN4 9FH
MADE IN CHINA

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MH Handel GmbH