9

STEPS

20 MIN

EN_ IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

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INTRODUCTION

The rollators are manufactured from high-quality materials and include a seat, bag, and locking brakes. The hand-bars are adjustable in height and the rollator folds easily for transport or storage.

• INTENDED USE

Designed to assist the user to walk, rest and carry small items of shopping for both indoor and outdoor use on flat surfaces by a single individual with limited mobility with a maximum weight as shown. The intended user should possess the necessary physical, visual, and cognitive skills to operate the rollator safely. If you have any doubt, please contact a qualified technician. It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the rollator or surrounding property.

• SAFETY ADVICE

This rollator is designed to be used as a mobility aid, creating a natural gate as the user walks securely with their primary weight between the handle bars with the unit in front of the user. Read this manual carefully before using your rollator.

• DO

- (1) Always ensure the brakes are in the locked position if the rollator is left unattended.
- (2) Check all castors and wheels are secure before use.
- (3) Check the brakes are functioning properly.
- (4) Check handle height adjustment hand knobs are secure before use.
- (5) Check the rollator is fully unfolded.
- (6) Take care when carrying the rollator when folded.
- (7) Keep your hands away from the folding mechanism to prevent any risk of pinching.

• DO NOT

- (1) Sit on the rollator if the brakes are not locked, this could cause the rollator to move resulting in injury to the user or other persons.
 - (2) Drop into the seat of the rollator, always sit into the rollator in a smooth, controlled motion. Seek assistance if necessary.
 - (3) Hanging objects on the brake levers or hand-grips, this could affect brake operation and/or stability of the rollator.
- Every care has been taken to ensure that your rollator will provide safe and reliable service as you adhere to the advice given above.

Figure 1

Your rollator is equipped with various components, you should know these components and parts before proceeding to read this user manual. Design and specifications could be changed without further notices.

SPECIFICATION

- Overall width: 63cm
- Overall depth: 68.5cm
- Seat area: 45*24.5cm
- Adjustable handle height: 83.5-98.5 cm
- Max. user weight :136 kg
- Net weight: 10 kg

1. Remove the rollator from the packing and remove the protective foils. The rollator will be folded as shown in Figure 1.

2. Take out the fastening knob that has been inserted on the front wheels tube, unfold the front wheels, then insert the fastening knob again into the front wheels tube and turn clockwise until it locks, as shown in Figure 2.

Figure 2

3. Pull up the quick-fold screw and unfold rear wheels as shown in Figure 3.

4. Press the both sides of the seat, then the seat was secured at a click.

Figure 5

5. Attach the bag. Hook the tabs of the bag into the hooks provided.

Brake


1. Pull the brake handles up to brake the rollator.
2. Push the brake handles down to lock the rear wheels. In this position, the parking brake of the rollator is active. To release the parking brake again, pull the brake handles upward.
3. Brakes must be applied in the lock position when the seat is going to be used. Precheck the lock by pushing forward on the rollator.

Handle Adjustment

The rollator's handles can be adjusted in height to suit different users. Simply depress the spring button and set it to the desired height. Repeat for the opposite side and make sure that both handles are secure and set at the same height before use.


Brake adjustment is carried out as follows: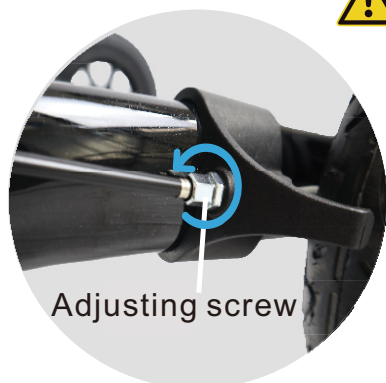

The brakes must always be adjusted so that they are the same on both wheels. Braking effect is achieved. If the rear brake is set too light or too tight, it will affect the function of the brake!



Turn the adjusting screw anti-clockwise to increase the tension, then lock the adjustment locking nut clockwise whilst holding the adjuster screw stationary. Brake adjustment reduced to allow light pressure may be ineffective on the brake position.

Removable cane holder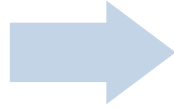

Attach the cane holder to the side of the frame in the provided position. To do this, slide the cane holder onto the holder from above.

Folding and storing the rollator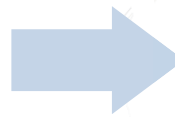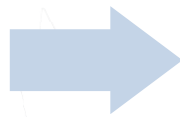


Step 1:
Pull the handle in the middle of the seat firmly upwards to fold the rollator.

Step 2:
Continue to push the rollator on both sides of the frame.

Step 3:
Pull up the quick-fold screw and fold that both rear wheels upwards.

Folding and storing the rollator

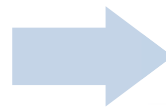Step 4:
Remove the screws and fold the front wheels upward .

Complete!

MAINTENANCE

Over a period of time, the brake cables may stretch, and if no further adjustments can be made, the cables should be replaced. Seek the help of the manufacture when replacing brake cables. The castors should be checked that they are secure each time before using the rollator. Also, check the brakes operate correctly and the handles and hand knobs are secure. Should any part of the rollator appear loose or insecure, discontinue use until it is inspected by a qualified technician.

The following shows the maintenance schedule:

Weekly:

- Check brake for malfunction.
- Check all wheels and castors for loose bolts and nuts and knobs.

Periodically:

- Check hand grips for wear
- Check frame bolts are secure
- Check front and rear axle bolts are secure
- Check tires for wear or cracks

CLEANING

Your rollator can be cleaned with a damp cloth or sponge and warm soapy water. Domestic anti-bacterial surface cleaners may be used if required. Care should be taken not to leave any cleaning residues on the hand grips or seat (where fitted) which may leave them slippery when used. Do not use a scouring pad for scrapers this may damage surface coatings.

WARNING!

1. Maximum working limit 136kg.
2. Do NOT stand on this product.
3. Ensure both push handles are adjusted to the same height.
4. Seek advice from a trained healthcare professional for height adjustment of this product.
5. Always maintain a safe distance between your rollator and other pedestrians or objects.
6. Some components may cause harm or damage when they come into contact with other objects.
7. Do not attempt to replace the wheels or castors of your rollator. If they are faulty or require a replacement, please contact qualified technician.
8. Ensure that your rollator is opened fully (folding scissor arm is straight) before reusing it.
9. Ensure that your fingers are clear from the scissor arm when folding.

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If you have any questions, please contact our customer care center, Our contact details are below:

US

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC
7777 Center Ave. Suite 430
Huntington Beach, CA 92647-9998 USA
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

UK

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:
MH STAR UK LTD
1 Northampton Cross Logistics Park
Northampton Cross Road
Northampton
Nn4 9FH
Tel: 0800 240 4050
enquiries@mhstar.co.uk
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

CA

 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.
Unit 33-1 Sparks Avenue, North York, Ontario,
Canada
M2H 2W1
MADE IN CHINA

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