
IN260200083V01

713-218V00

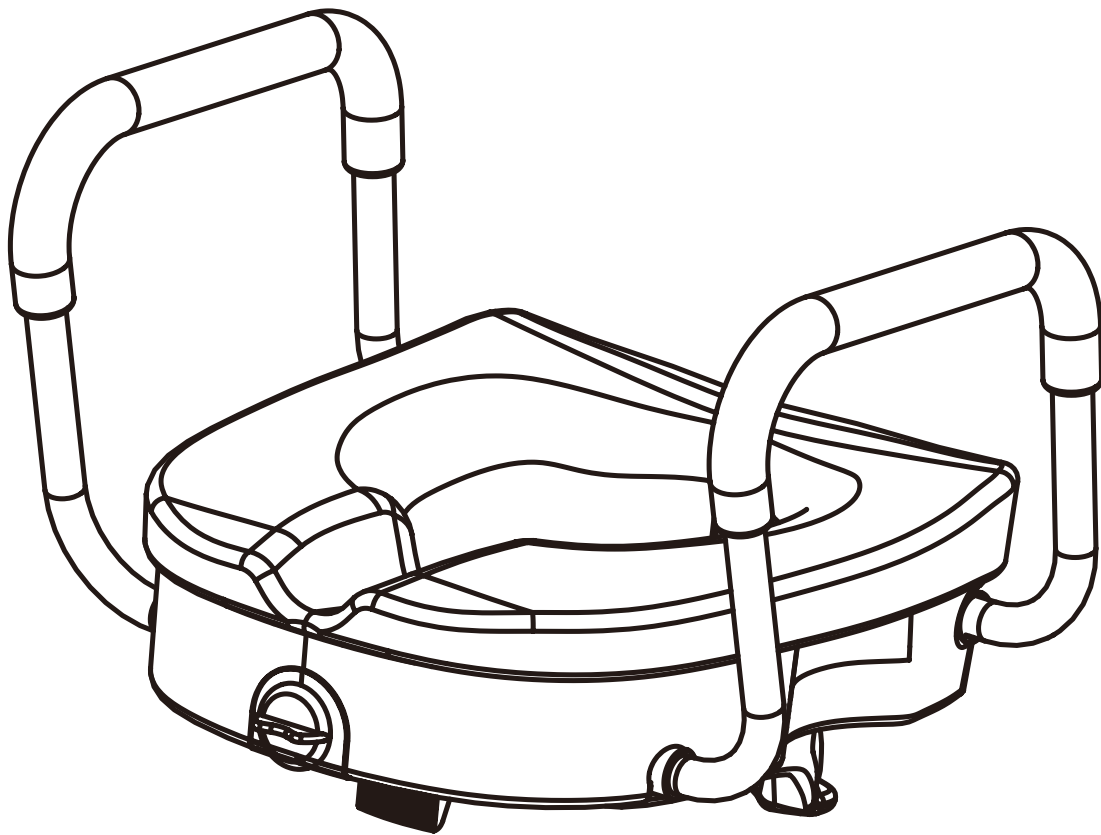3

STEPS

15 MIN

EN_ IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

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WARNING

DO NOT use if any parts were damaged in shipping.


Take care during assembly not to lose your balance when testing the connections. DO NOT use if poorly attached.

ALWAYS take care to remain balanced while sitting and rising.

FOR USE ONLY BY PERSONS WEIGHING LESS THAN 136 KG.

MAINTENANCE AND CARE


- Use mild detergent or household cleaner with warm water and wipe dry.
- **IMPORTANT:** DO not use abrasive cleaners or bleach to clean the product.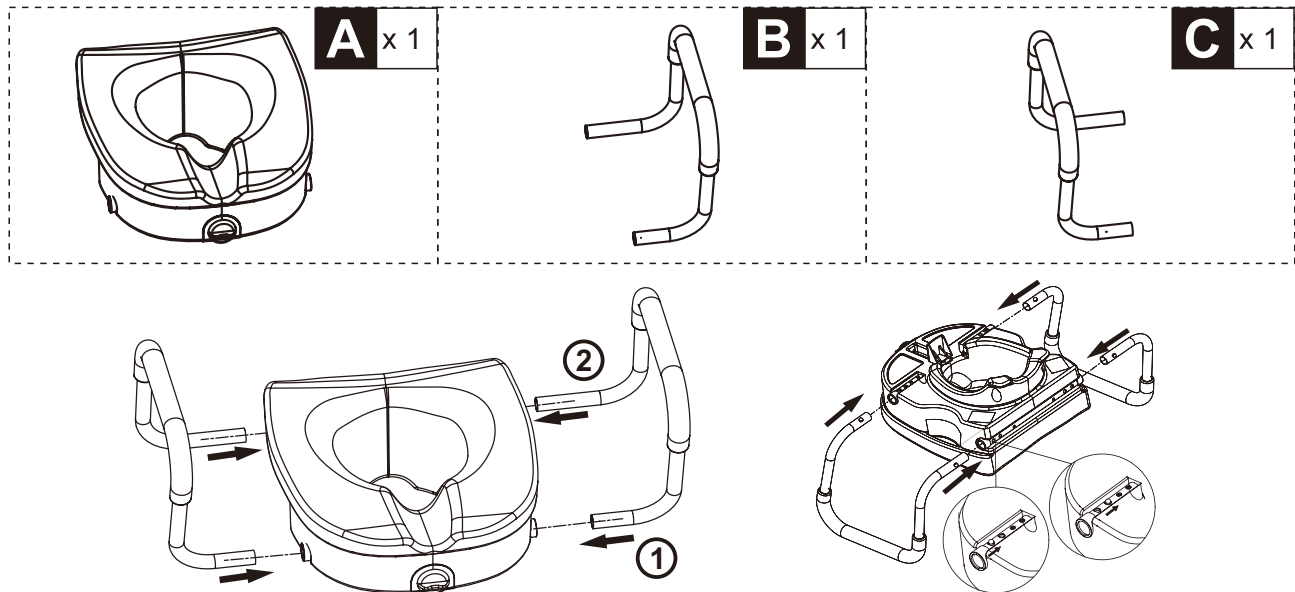


1. To install arms, depress push pins on arms and insert into mounting tubes. The shorter tube (1) inserts in front.

There will be an audible click. Ensure the push pins fully protrude through the holes in the mounting tube. Make sure arms are locked in place by pulling them. The hand rail distance can be adjusted closest to the seat by locking onto the innermost locking holes. The hand rails can be farther from the seat by using the outermost locking holes.

To remove arms, reverse step.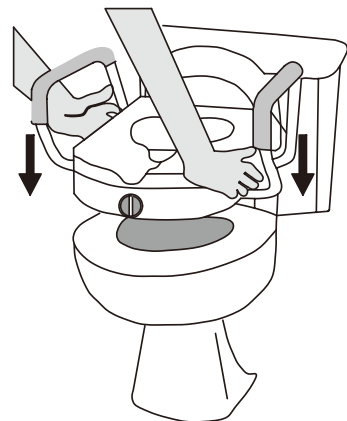

2. Lift toilet lid assembly and place rear wings of toilet seat around rim of toilet bowl. (Some toilet bowls may require that the original seat assembly on the toilet be removed to allow proper fit.) In addition the front adjustment bracket must clear the rim in order for the raised toilet to fit securely.

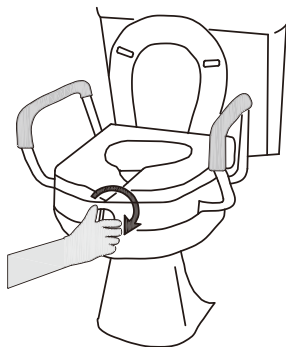3. With raised toilet seat resting correctly on rim, straighten and center product over toilet bowl as necessary. Turn adjusting knob clockwise to tighten product securely in place. Hand tighten only use no tools, and be careful not to over tighten, as threads on adjustment bracket may strip. Check for secure fit before using product. Periodically check the raised toilet seat to ensure it is not loose, if loose, tighten the adjusting knob until raised toilet seat is secure. User's weight should be centered over toilet seat, not too far forward or too far to one side, as unit may tip.

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If you have any questions, please contact our customer care center, Our contact details are below:

US

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC
7777 Center Ave. Suite 430
Huntington Beach, CA 92647-9998 USA

If you have any questions, please contact our customer care center, Our contact details are below:

CA

 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.
7270 Woodbine Avenue, Unit 307, Markham,
Ontario Canada L3R 4B9
MADE IN CHINA

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If you have any questions, please contact our customer care center, Our contact details are below:

UK

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:
MH STAR UK Ltd
1 Northampton Cross Logistics Park
Northampton Cross Road
Northampton
NN4 9FH
MADE IN CHINA

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MH Handel GmbH