**EN\_ IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.**

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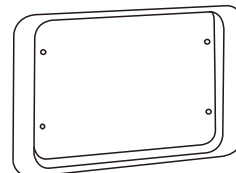
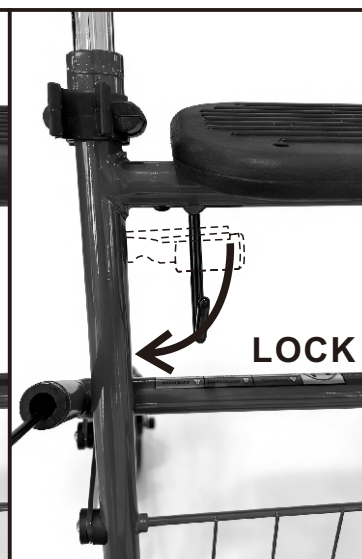
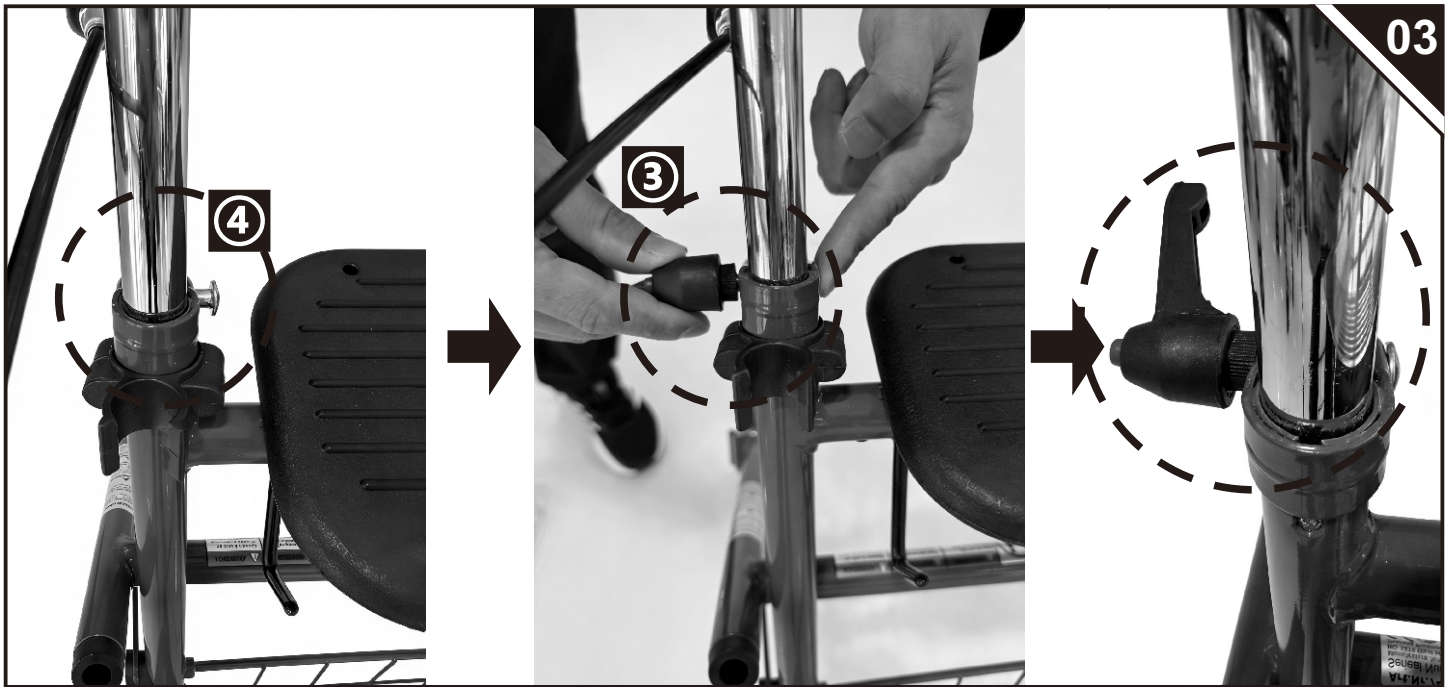
①

Fold

1. Engage the brake lock
2. Release the safety lever
3. Pull the center tube backward and tilt it
4. Fold down

Open

1. Stand up
2. Push the rear wheel backward/push the seat downward
3. Lock the safety lever (Safe to use only when locked)

Check the parts for any looseness, wear, or damage periodically. If any problems are found, correct them before using them.

1. The exterior of this rollator walker should be cleaned periodically with a soft damp cloth. Do not use abrasive cleaners or caustic chemicals.

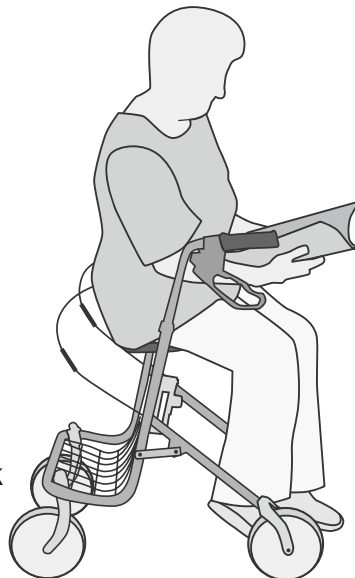
Sit down.

Lock

Unlock

Set height

The seat is only intended for brief rest periods and should not be used as a chair for extended periods.

When braking, operate both brakes simultaneously.

Set height

1. Loosen the fixing screws on both sides
2. After adjusting the height, tighten the screws
3. Adjust both sides to the same height! (Recommended height: Handle level with wrist joint)
4. Pull the fixing screw rod backward, then adjust the fixing screw once it's parallel to the frame

Installing Accessories

Place the board on the seat (the raised edge of the board must be inserted into the seat position).

Hang the basket on the hook (Remove the basket before storing the walker!)

Only suitable for transporting smaller items. Maximum weight: 5 kilograms!

Remember to remove the board before sitting down!

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If you have any questions, please contact our customer care center, Our contact details are below:

**US**

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC  
7777 Center Ave. Suite 430  
Huntington Beach, CA 92647-9998 USA

If you have any questions, please contact our customer care center, Our contact details are below:

**CA**

 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.  
7270 Woodbine Avenue, Unit 307, Markham,  
Ontario Canada L3R 4B9  
MADE IN CHINA

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If you have any questions, please contact our customer care center, Our contact details are below:

**UK**

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:  
MH STAR UK Ltd  
1 Northampton Cross Logistics Park  
Northampton Cross Road  
Northampton  
NN4 9FH  
MADE IN CHINA

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MH Handel GmbH