EN_ IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

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


Integrated Handles to assist in standing and sitting

- Height adjustable legs from 14" – 21" for proper fit
- Soft padded cover for comfort
- Padded cover pops in and out for easy cleaning
- Seat and pad have drainage holes to shed water
- 136 KG weight capacity
- No-tools easy assembly
- Fits most standard bathtubs, showers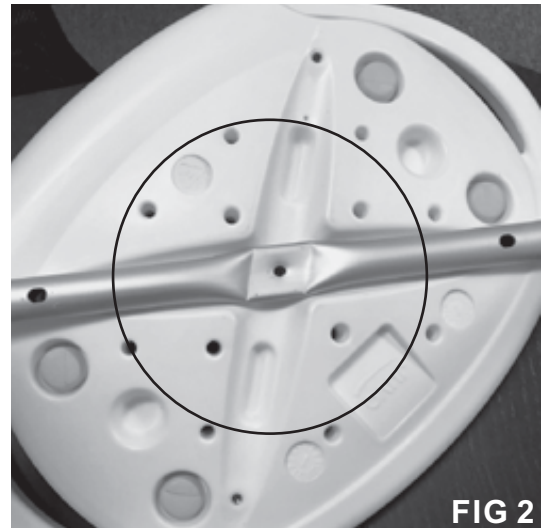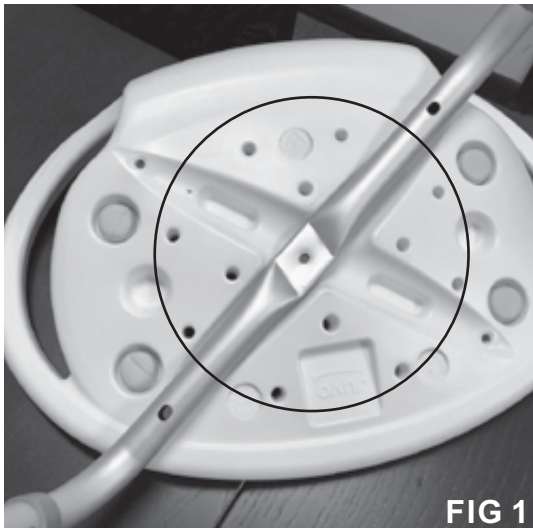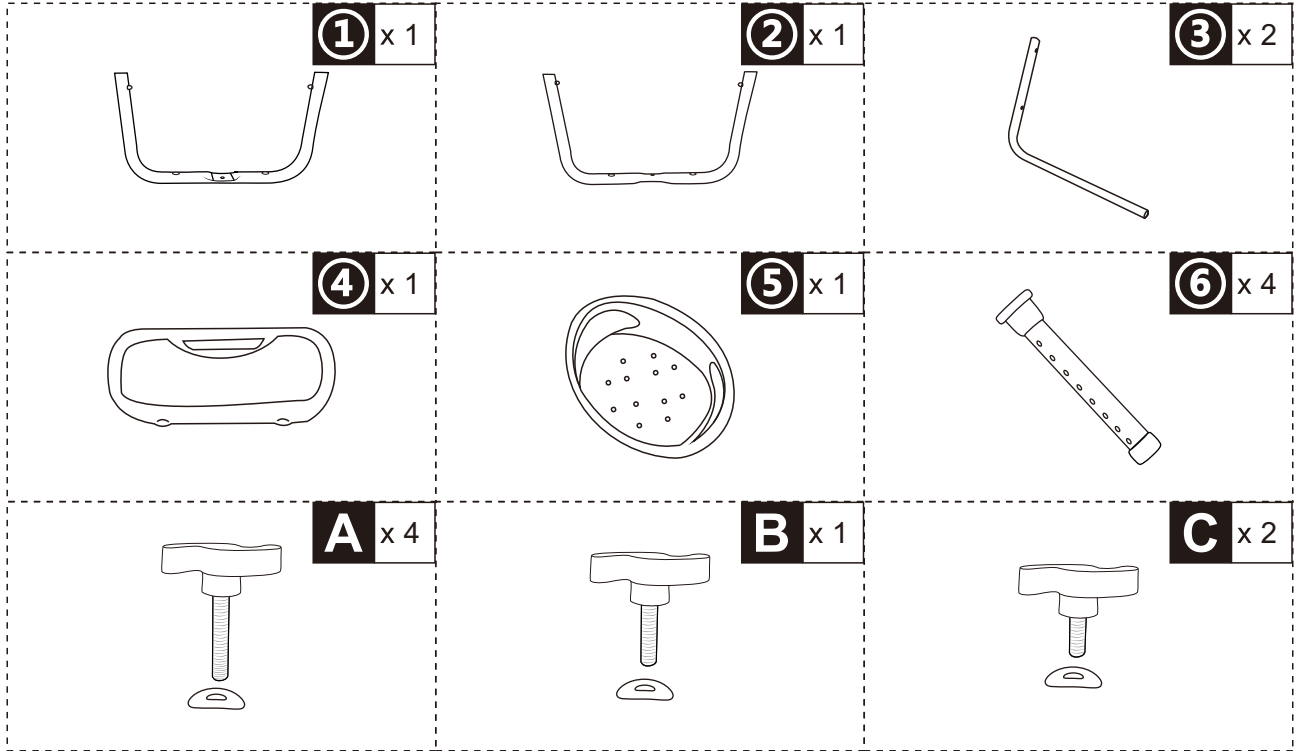

1. Turn Seat Upside Down

- Place seat on flat surface i.e. table or floor, upside down so underside is facing you

2. Connect First U-Shaped Leg Brace

- Lay U-Shaped Leg Brace #1 across underside of seat. Make sure it is resting in recessed channel, marked #1, such that the flat portion of the tubing is aligned with the direction of the channel. IMPORTANT.(Refer to FIG 1 and 2)

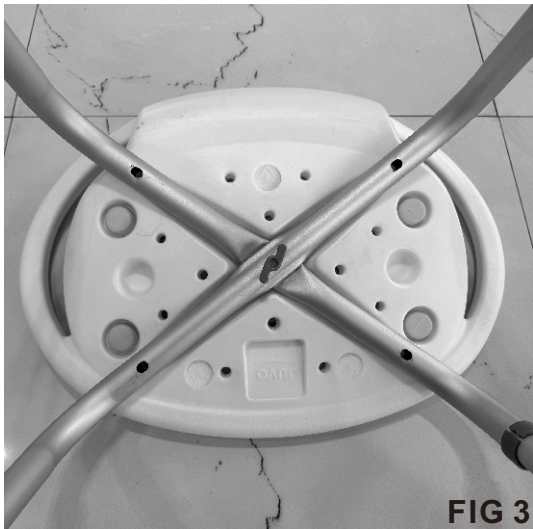
FIG 3

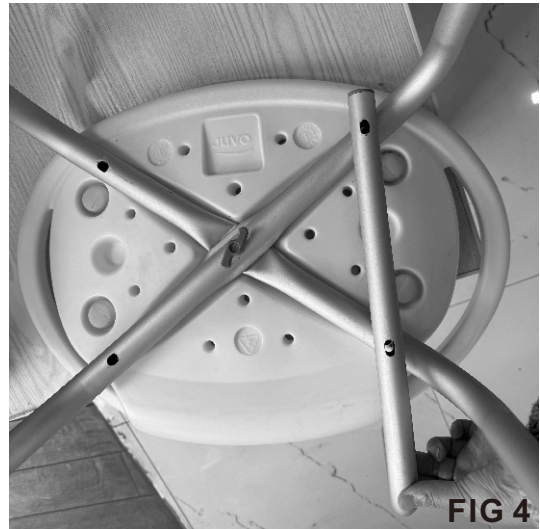
FIG 4

3. Connect Second U-Shaped Leg Brace

- Lay U-Shaped Leg Brace #2 on top of, across U-Shaped Leg Brace #1, in recessed channel marked #2. (Refer to FIG 3)
- Fasten U-Shaped Leg Braces #1 and #2 to underside of seat with Butterfly Bolt with Washer starting in the center first. Use Butterfly Bolt with Washer – Medium to fasten center position first. Do not over tighten

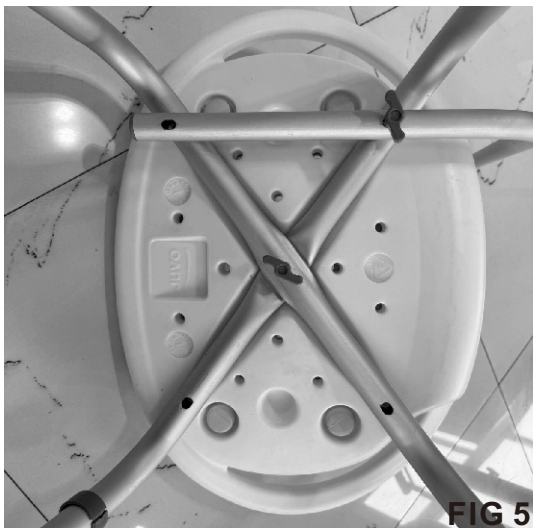
FIG 5

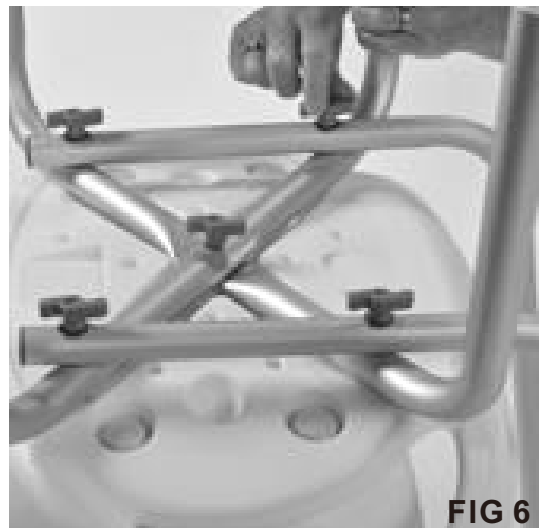
FIG 6

4. Connect Back-Rest Support Bars and U-Shaped Leg Braces

to Underside of Seat- Lay one Back-Rest Support Bar across the U-Shaped Leg Braces such that the two holes of the Back-Rest Support Bar align with the two holes of the U-Shaped Leg Braces, (refer to FIG 4). Make sure that the Back-Rest Support Bar is facing up, from the back side of the chair.

- Fasten Back-Rest Support Bar and U-Shaped Leg Brace to underside of chair with two Butterfly Bolts with Washers – Long. (Refer to FIG 5). Do not over tighten.
- Repeat process of fastening Back-Rest Support Bar and U-Shaped Leg Brace to underside of chair – with second Back-Rest Support Bar. (Refer to FIG 6).

FIG 7

FIG 8

5. Connect Padded Back Rest to Back-Rest Support Bars

- Turn Chair right side up.
- Insert two Back-Rest Support Bars into Padded Back Rest channels.
- Fasten Padded Back Rest to Back-Rest Support Bars with two Butterfly Bolts with Washers – Short. (Refer to FIG 7).

6. Tighten All Butterfly Bolts with Washers

- Make sure all seven Butterfly Bolts with Washers are firmly fastened. Do not over tighten. (Refer to FIG 8).

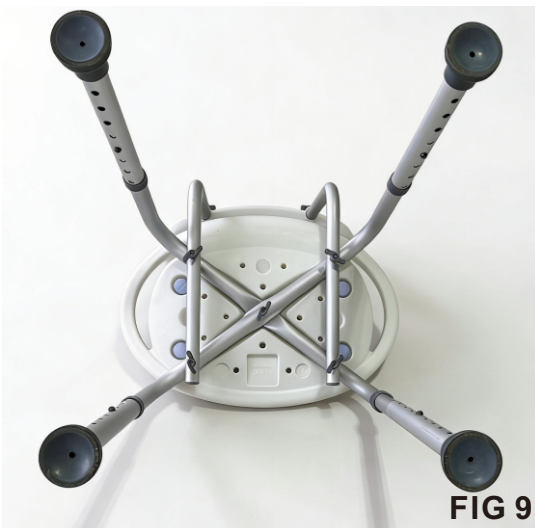
FIG 9

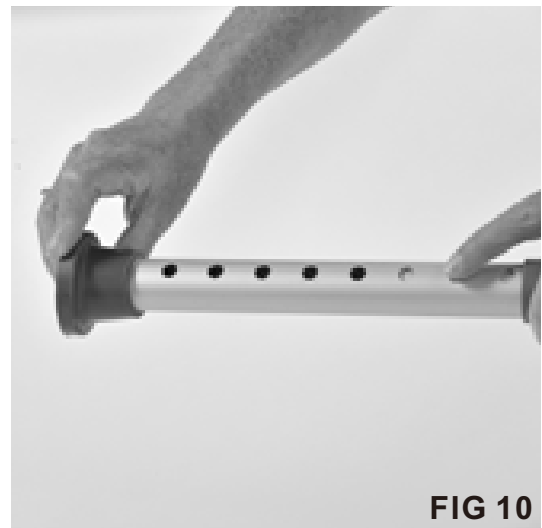
FIG 10

7. Connect Adjustable Legs with Rubber Foot to U-Shaped Leg Braces

- Take first Adjustable Leg with Rubber Foot and connect it to U-Shaped Leg Brace, (Refer to FIG 9). Using thumb, push down on the push button on the U-Shaped Leg Brace. The Adjustable Leg with Rubber Foot will then slide onto the U-Shaped Leg Brace, (Refer to FIG 10)
- Slide Adjustable Leg with Rubber Foot onto U-Shaped Leg Brace to desired height position. Push down on push button as needed
- IMPORTANT – Make sure that push button is fully engaged with the Adjustable Leg with Rubber Foot. The push button should click into place in the desired height position and protrude through the hole
- Repeat the process for the remaining three Adjustable Legs with Rubber Feet

SAFETY PRECAUTIONS - READ ALL INSTRUCTIONS BEFORE USE

1. Make sure all four Adjustable Legs with Rubber Feet are securely connected to U-Shaped leg braces at the same height adjustable position. Each push pin must click into place, be fully visible through the preferred height adjustable hole.
2. Make sure all fastening positions with Butterfly Bolts with Washers are firmly fastened. Do not over tighten
3. Make sure all four rubber feet are flat on the inside floor of the bathtub or shower
4. Test Bath Safety Chair for stability before applying full weight

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If you have any questions, please contact our customer care center, Our contact details are below:

US

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC
7777 Center Ave. Suite 430
Huntington Beach, CA 92647-9998 USA
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

CA

 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.
7270 Woodbine Avenue, Unit 307, Markham,
Ontario Canada L3R 4B9
MADE IN CHINA

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If you have any questions, please contact our customer care center, Our contact details are below:

UK

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:
MH STAR UK Ltd
1 Northampton Cross Logistics Park
Northampton Cross Road
Northampton
NN4 9FH
MADE IN CHINA

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MH Handel GmbH