4
STEPS

20 MIN

EN_ IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

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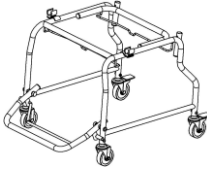
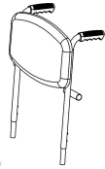
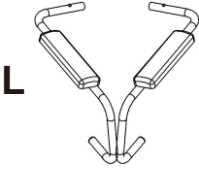
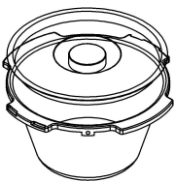
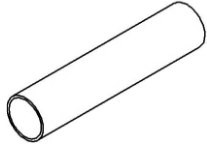
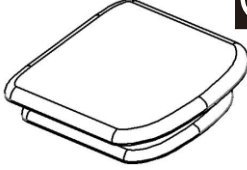
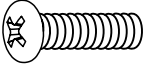
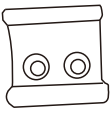
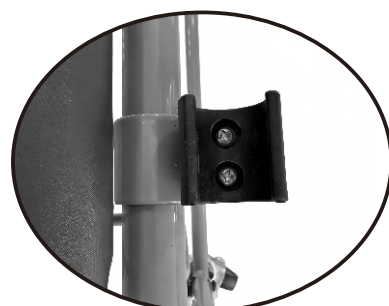
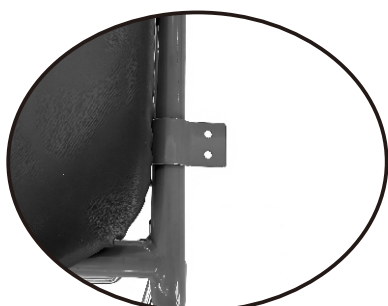
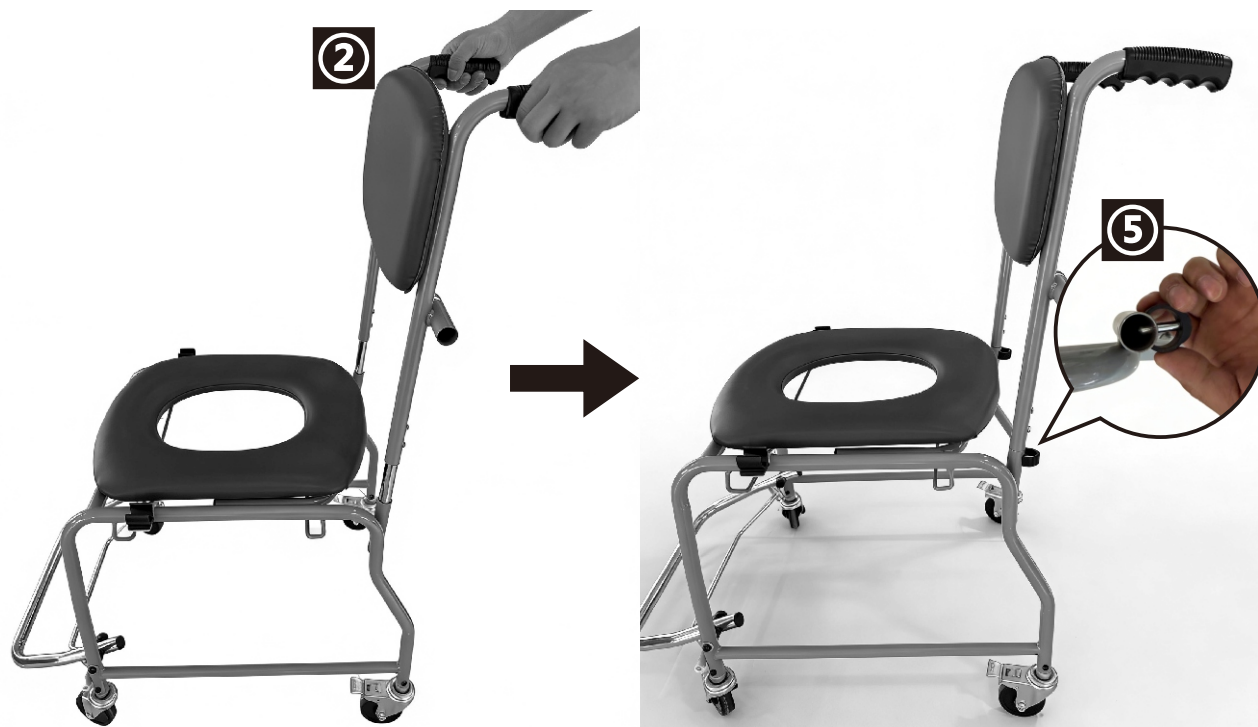
EN_ Open the package, remove the main frame of the commode chair and place it on a level surface, Align the steel tube of the commode chair's backrest with the steel tube which above the main frame of commode chair; Press down until inserting into the commode chair's main frame. Take two plastic parts (5.E-type clips) and align them with the steel tube holes on the rear side of the main frame to Make the backrest fixedly linked with the main frame.

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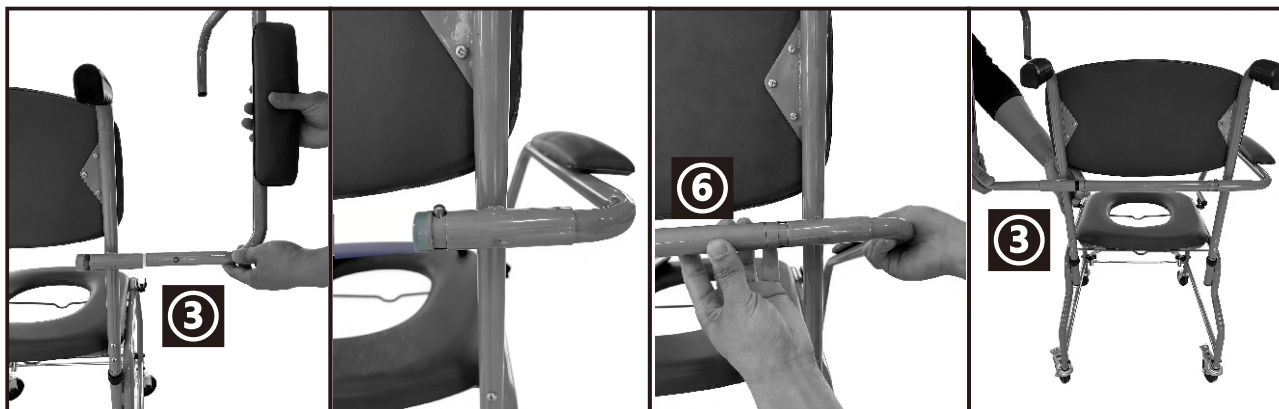
EN_ Take out the left and right armrests separately, Press the copper jump beads and symmetrically insert them into the steel tube behind the backrest of the toilet chair, At the same time, take (6. Handrail support tube) and place it in the middle of the backrest steel tube, allowing the jump beads on the left and right handrails to pop out and lock. Then flip the left and right handrails up and down to align the front end of the tube with the plastic clip above the main frame.

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EN_ Insert the toilet bucket component from both sides into the toilet bucket bracket of the commode chair, then rotate it to the correct position so that the handles on top of the toilet bucket do not interfere with each other. Remove the lower soft pad, position the plastic part to fit securely over the horizontal bar at the rear of the commode chair main frame, allowing it to flip up and down; then place the upper soft pad on top of the lower soft pad.

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EN_ Other features: The accompanying bucket and upper cushion can be removed. The commode chair also can be moved above home toilet seat, with User weight within 136kg; It can also be temporarily used as a caretaker wheelchair on flat indoor surfaces; Flip up the left and right armrests to facilitate the transportation of family members and users.

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MH Handel GmbH